# The modern ward bed with intelligent thinking...









The modern ward bed with intelligent thinking...



# Designed around... patient safety

Building on over half a century of both patients and staff and increased

Coupled with this the new ward bed is easy to clean and maintain, and is meeting the broadest spectrum of clinical needs.

This level of design flexibility means patient groups, resulting in reduced requirements for bed to bed transfer and increased relevance for the entire

Low height reduces risks associated with

**T** +44 (0) 1422 233 000 **F** +44 (0) 1422 233 010 E sales@sidhil.com W www.sidhil.com 3

## Designed around... identified clinical needs

Sidhil's all new iQ Contour enables the backrest and knee break to operate in harmony to reduce shear and friction, reducing the risk of tissue damage and improving patient comfort.



With the Innov8 iQ, Sidhil has successfully brought together an in-depth understanding of the changing requirements of the acute sector with state of the art manufacturing and assembly methods. This innovative ward bed achieves the optimum combination of safety, performance, functionality and cost efficiency.

The auto regressing backrest reduces patient movement when the backrest is being raised or lowered. The backrest also features an anti-trapping ratchet actuator which prevents the backrest dropping following any resistance, until the carer is ready to move the bed again safely.

### Cardiac chair position



The simultaneous operation of the cardiac chair and tilt functions combined with auto regression improves patient positioning. mobilisation and rehabilitation and reduces risk of pressure damage.

## Designed around... infection control







Patient handset innovative design to ensure accessible location of handset for patient.







Movable catheter hooks can be placed



Integral bed extension combines ease of use




Low height foot end to of colour options for ward

**T** +44 (0) 1422 233 000 **F** +44 (0) 1422 233 010

## Designed around... ease of use

Nurse Handset Functions

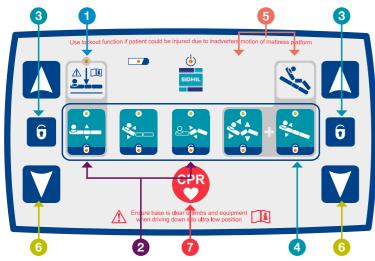

NEW Nurse Handset Holder now available as an accessory

#### 1 Low height light on handset

The light automatically activates when the bed is being operated in the low height region, to alert the carer to potential risks.

#### 2 Platform height, backrest & leg section functions

The Innov8 iQ is a fully electric profiling bed. Using either the patient or carer handset the mattress platform height, the auto regressing backrest and leg section can easily move to ensure patient comfort and safety.

#### 3 Function lock

One or more handset functions can be locked so unintentional movement does not harm or cause discomfort to the patient.

#### 4 Tilt (Trendelenburg)

Carer can set mattress platform angle between -13° and +13°. Inbuilt software avoids entrapment and clash points with individuals and equipment.

#### 6 Cardiac chair

Cardiac chair function allows simultaneous operation of all profiling sections and tilt function, providing a comfortable seated position in bed.

#### 6 Ambidextrous operation

Carer handset can be operated with both left and right hands to suit individual preferences or switching between hands where required.

7 CPR Function

#### **Brake Bar Functions**

Brake bar/pedal ensures easy access from foot or side of the bed.

#### Triple function castors enable





locked.



Free all four castors free to roll for easy manoeuvrability between in limited spaces.

Track for easy bed movement around and wards.



Extremely light weight bed ensures ease of manoeuvrability with minimal effort.



The built in diagnostics port allows for swift and easy maintenance procedures.



The Nurse Handset can now be clipped neatly away into our NEW Nurse Handset Holder

## The Innov8 iQ split side rails



incorporating split side rails to help protect against the risk of falls.

Satisfying the highest standards for safety and reliability and designed to meet the hospital environment.

Split side rails lower to below the level of the mattress to help patients get in and out of bed safely, with self-lowering function for ease of use.

#### Innov8 iQ Bed now available with X-Ray Backrest:

- X-Ray translucent backrest (includes
- Smooth action, easy to use
- shatter resistant
- Corrosion resistant, easy-clean stainless
- Fully adjustable to fit all universal
- X-Ray cartridges
- Suitable for use with C-arm X-Ray
- Factory fitted option, compatible with all

Bed Code	Code
Innov8 iQ Bed	IQ/AQ
Innov8 iQ Bed with standard side rails	IQ/AQ/SR
Innov8 iQ Bed with high side rails	IQ/AQ/HSR
Innov8 iQ Bed with split sides rails	IQ/AQ/SSR
Innov8 iQ Bed with split sides rails and x-ray backrest	IQ/AQ/SSR/ XB
Innov8 iQ Bed with standard sides rails and x-ray backrest	IQ/AQ/SR/ XB
Innov8 iQ Bed with high sides rails and x-ray backrest	IQ/AQ/HSR/ XB

Bed Dimensions		
Mattress Platform Length		200cm
Mattress Platform Width		88cm
Outside Bed Length	- Standard	228cm
	- Extended	248cm
Outside Bed Width		103cm
Mattress Platform	- Maximum Height	75cm
Mattress Platform	- Minimum Height	34cm
Panel Base	- Minimum Height	29.3cm
Bed Weight (with standard side rails)		130kg
Safe Working Load		255kg (40st)
Maximum User Weight		190kg

General Accessories	Code
Innov8 Lifting Pole	INNOV8/LP
Innov8 IV Pole	INNOV8/RANGE/DP
Innov8 Traction - Balkan Beam	TRACT/BB/KIT
Innov8 Traction - Foot End	TRACT/FE/KIT
Oxygen Cylinder Carrier	INNOV8/RANGE/OC
Innov8 Range Handset holder	INNOV8/RANGE/HSH

## Service centres nationwide

With service centres strategically located nationwide, we guarantee an optimum response, supported by our team of field service engineers. Their experience and depth of product knowledge ensures equipment is put back into service promptly, efficiently and cost effectively.

We carry comprehensive stocks of spare parts both for the Sidhil range and for other manufacturers' products at our service centres. We also provide decontamination services to protocols as advised by a clinical scientist in infection control, staffed by trained personnel operating in accordance with MHRA DB2006 (05).

Our service centre processes are embedded within Sidhil's established quality standards to ISO 9001 levels.

For more information on our service and maintenance capabilities, contact the Service Team on: 01422 233 136.



- Glasgow
- Sunderland
- Halifax
- Warrington
- Kettering
- Orpington
- N. Ireland

#### Service & Maintenance & Rentals

Product Servicing - All Makes / Decontamination / Spares Short term solutions available as an alternative to outright purchase





Sidhil Business Park Holmfield, Halifax, West Yorkshire HX2 9TN

Tel: +44 (0) 1422 233000 Fax: +44 (0) 1422 233010 Email: sales@sidhil.com Web: www.sidhil.com







Independently verified to IEC 60601-1 and IEC 60601-2-52 through a 3rd party UKAS approved test house